Instruction for Use

Oxygen Administration Tubing: Link (Bubble) Tubing

Pennine Oxygen administration tubes are non-sterile, single use devices intended to provide an additional length of tubing between the oxygen source and patient oxygen delivery device, in patients requiring supplementary oxygen from adults to paediatrics in both clinical, ambulatory, and home care environments. To be used by trained clinicians / homecare user, in clinical, homecare or emergency settings.

Instructions for Use:

Caution: FOR SINGLE USE, DO NOT REUSE, DO NOT STERILISE, NON-STERILE DEVICE.

- Dispense the length of tubing required, pulling the required length from the centre of the coil.
- Cut across the bubble to suit the connection required.
- With the oxygen source switched off, push fit the connector end to the oxygen source.
- Push fit the opposite connector end onto the oxygen delivery device whose connectors conform to ISO 13544-2:2002 +A1:2009.
- Ensure oxygen administration tubing is securely connected to connectors.
- If you experience problems using the device contact your healthcare professional for advice. Serious incidents that occur in relation to the devices intended use, should be reported to the manufacturer and local competent authority.

Warnings:

- Oxygen is a non-flammable gas, but does support combustion, please take note of warnings supplied with your oxygen source equipment.
- Do not use the oxygen tubing if you observe a blockage in the tubing as this may reduce or prevent oxygen administration.
- Do not use the oxygen tubing if you observe any fluid or foreign bodies in the tubing as this may reduce or prevent oxygen administration or cause infection to patient.
- Do not kink or tie your oxygen tubing as this may reduce or prevent oxygen administration.
- Do not place anything on your tubing that may obstruct flow this includes clothing, bedding etc. as this may reduce or prevent oxygen administration.
- Keep excess oxygen tubing loosely coiled and out of the way to prevent trips and falls that might result in injury to the patient, user or other.
- Do not use for longer than 30 days as the device may not perform as intended.
- Suitable for flow meter range for medical oxygen between 0-15 litre per minute. Use outside of this flow range may result in the device not performing as intended.
- Do not use the device past its expiry date as the device may not perform as intended.
- Do not use if damage observed to packaging or product as the device may not perform as intended or may cause infection to the patient, user or other.
- Device is intended for single patient use.
- Do not reuse device as there may be a risk of infection to patient, user or other and the device may not perform as intended.

Storage and Disposal:

EN

- Store in a clean, cool, dry place away from direct sunlight.
- Dispose of as clinical waste in-line with local and Government guidelines.

Ivor Shaw Ltd Trading as Pennine Healthcare
300 City Gate, London Road, Derby, DE24 8WY
Tel: +44 (0)1332 794880 Fax: +44 (0)1332 794890
www.penninehealthcare.co.uk

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