

Intermittent Urethral Catheter – Safety and Performance Information

Warnings:

Caution: FOR SINGLE USE, DO NOT REUSE, STERILE DEVICE.

- Do not use if packaging is damaged or open, sterility will be compromised, there may be a risk of infection and the device may not perform as intended.
- Do not use if you observe any damage to, blockage of, or foreign bodies in the device as the device may not perform as intended or cause injury to the patient, user or other.
- Do not use if side eyes are missing as the device may not perform as intended.
- Do not use if you observe any kinking in the tubing as this may reduce suction and the device may not perform as intended.
- Do not use the device past its expiry date. As it may impact the performance of the device and there may be a risk of infection
- Device is intended for single patient use.
- Do not reuse device as there may be a risk of infection to patient, user or other and the device may not perform as intended.
- Device is intended and designed for intermittent use only. Failure to remove the device after bladder drainage may result in infection.
- If you experience problems using the device, contact your healthcare professional for advice. Serious incidents that occur in relation to the devices intended use, should be reported to the manufacturer and local competent authority.

Storage and Disposal:

- Store in a clean, cool, dry place away from direct sunlight.
- Dispose of as clinical waste in-line with local and Government guidelines.