

## Oxygen Tubing – Safety and Performance Information

### **Warnings:**

- Oxygen is a non-flammable gas, but does support combustion, please take note of warnings supplied with your oxygen source equipment.
- Do not use the oxygen tubing if you observe a blockage in the tubing as this may reduce or prevent oxygen administration.
- Do not use the oxygen tubing if you observe any fluid or foreign bodies in the tubing as this may reduce or prevent oxygen administration or cause infection to patient.
- Do not kink or tie your oxygen tubing as this may reduce or prevent oxygen administration.
- Do not place anything on your tubing that may obstruct flow this includes clothing, bedding etc. as this may reduce or prevent oxygen administration.
- Keep excess oxygen tubing loosely coiled and out of the way to prevent trips and falls that might result in injury to the patient, user or other.
- Do not use for longer than 30 days as the device may not perform as intended.
- Suitable for flow meter range for medical oxygen between 0-15 litre per minute. Use outside of this flow range may result in the device not performing as intended.
- Do not use the device past its expiry date as the device may not perform as intended.
- Do not use if damage observed to packaging or product as the device may not perform as intended or may cause infection to the patient, user or other.
- Device is intended for single patient use.
- Do not reuse device as there may be a risk of infection to patient, user or other and the device may not perform as intended.

### **Storage and Disposal:**

- Store in a clean, cool, dry place away from direct sunlight.
- Dispose of as clinical waste in-line with local and Government guidelines.