

Sterile Suction Connecting Tube – Safety and Performance Information

Warnings:

- Do not use if packaging is damaged or open, sterility will be compromised, there may be a risk of infection and the device may not perform as intended.
- Do not use if you observe any damage to, blockage of or foreign bodies in the device as the device may not perform as intended or cause injury to the patient, user or other.
- Do not use if you observe any kinking in the tubing as this may reduce suction and the device may not perform as intended.
- Device is intended for transient use. Do not use continuously for longer than 60 minutes as the device may not perform as intended.
- Do not use the device past its expiry date. As it may impact the performance of the device.
- Only to be used by clinicians or trained homecare users, in clinical, homecare and emergency settings.
- For use in removal of substances intended for disposal. Not for use with substance intended to (re)administered into the body.
- Do not use in conjunction with flammable gasses.
- If you experience problems using the device, contact your healthcare professional for advice. Serious incidents that occur in relation to the devices intended use, should be reported to the manufacturer and local competent authority.
- Device is intended for single patient use.
- Do not reuse device as there may be a risk of infection to patient, user or other and the device may not perform as intended.

Storage and Disposal:

- Store in a clean, cool, dry place away from direct sunlight.
- Dispose of as clinical waste in-line with local and Government guidelines.