

Surgical Suction Probes – Safety and Performance Information

Warnings:

- Pennine Surgical Suction Probes devices should not come into direct contact with the heart or the central circulatory system (CCS) or the central nervous system (CNS) as it is not intended for this application and may cause injury to the patient.
- Do not use if damage observed to packaging or product or if packaging is open prior to use as the device as it compromises the sterility of the device and/or may not perform as intended or cause infection to the patient, user or other.
- Do not use if you observe any damage to, blockage or foreign bodies in the device as the device may not perform as intended or cause injury to the patient, user or other.
- For Suction Probe sets do not use if you observe any kinking in the suction tubing as this may reduce the suction of the device therefore not performing as intended.
- Keep excess suction tubing loosely coiled and out of the way to prevent trips and falls and this may cause injury to the patient, user or other.
- Device is intended for transient use. Do not use for longer than 60 minutes as the device may not perform as intended.
- Do not use the device past its expiry date as the device may not perform as intended.
- Device is intended for single patient use.
- Do not reuse device as there may be a risk of infection to patient, user or other and the device may not perform as intended.

Storage and Disposal:

- Store in a clean, cool, dry place away from direct sunlight.
- Dispose of as clinical waste in-line with local and Government guidelines.